OneStep | Quick Start Guide: Setup

Foot Scanner™

Included in the box: **A** Cellular Router + Power Cord **B** OneStep Foot Scanner™ + Power Cord

Step 1 | Plug in the Router

Connect the power cord to the cellular router and plug in near where you plan to keep your OneStep Foot Scanner™

Step 2 | Plug in the Scanner

Place your OneStep™ Foot Scanner on a firm surface away from direct sunlight or other sources of heat that might impact the temperature analysis of your feet. Connect the power cord to the scanner and plug in.

When you see

READY

or

**STEP ON
DEVICE**

It's time to scan!

See back for scan help and troubleshooting

Bluedrop Medical

OneStep | Quick Start Guide: Scanning

Foot Scanner™


Before You Scan

If you are comfortable standing on the scanner, you may use as a normal scale. If not, we recommend using seated and applying slight pressure by leaning forward.

How to Scan



- Place your feet on the scanner within the scan zones.
- The scanner will automatically start when it senses you.
- Remain still until the scanner screen indicates the scan is complete.

Always leave your router and scanner plugged in as scans may not transmit immediately!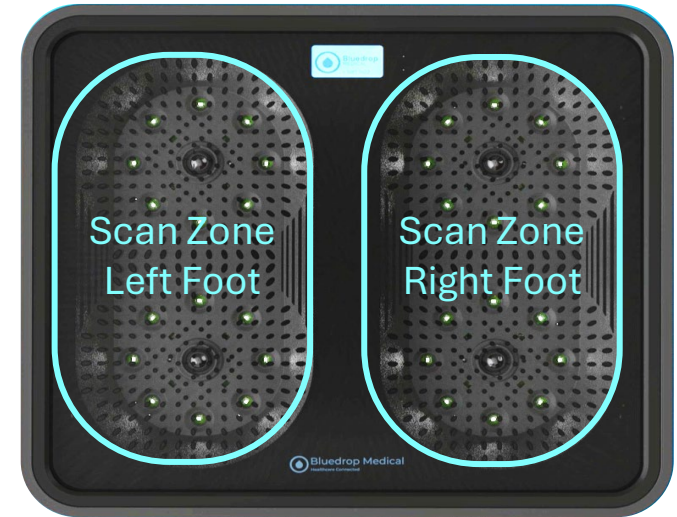

Troubleshooting

There may be times when the router and / or scanner lose connection. If you see one of the following messages on the scanner screen, we recommend unplugging both the router and scanner and waiting a few minutes. When plugging back in, plug the router in first, then the scanner. If this doesn't resolve the problem, reach out to your Bluedrop Specialist or call our general help line (855) 453-8707.

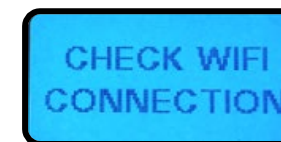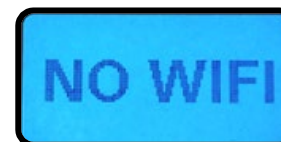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